THUASNE

CICATREX®

D: Narbenkompressionsbekleidung  
GB/USA: Scar compression garment  
F: Vêtement compressif de cicatrisation  
I: Indumenti di compressione per cicatrici  
E: Prendas de compresión para cicatrices  
P: Vestuário de compressão para cicatrizes  
NL: Compressiekleding voor littekens  
DK: Kompressionstøj til arbehandling  
S: Ärrkompressionsplagg  
FIN: Painevaatteet arpihoitoon  
N: Arrkompresjonsbekledning  
GR: Συμπιεστικό ένδυμα για ουλές  
CZ: Kompresivní prádlo pro terapii jizev  
PL: Odzież kompresyjna do blizn  
EST: Kompressioonriietus armide raviks  
LV: Rētu kompresijas apģērbi  
LT: Kompresinė apranga randams gydyti  
SK: Kompresívne odevy na liečbu jaziev  
SLO: Kompresijska oblačila za brazgotine  
H: Hegterápiás kompressziós ruházat  
RO: Îmbrăcămintă compresivă pentru cicatrici  
BG: Компресионно облекло при белези  
HR: Kompresijska odjeća za ožiljke  
RUS: Изделия из компрессионного трикотажа для лечения рубцов  
TR: Yara izi kompresyon kıyafeti  
ROK: 흉터 압박복  
J: 瘢痕治療用圧着衣  
AR: ملابس ضاغطة للندوب

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## Purpose

Compression textiles are basic elements in scar tissue therapy. The objective is to obtain flat and elastic scar tissue which does not reduce the quality of life.

With continuously planar pressure, CICATREX® compensates for the resilience and surface elasticity of the skin. Surface elasticity presents the natural limit for scarring to prevent the scar rising above the level of the skin. Blood circulation in the scar will be reduced, collagen fibres are aligned in parallel. CICATREX® is hypoallergenic, prevents inflammations in the tissue, relieves itching and provides comprehensive protection for the fresh scar.

The combination of continuous surface pressure of the compression garment with silicone has a positive effect on scar maturation, and the scar area will be visually and functionally improved. For this combination, Silon-TEX® can be sewn into the CICATREX® compression garment. Silon-TEX® is a soft and supple textile carrier material on which a thin medical silicone layer is applied. Medical silicone provides a moist environment on the scar. The scar will thus not dry out. It keeps the scar soft and has a positive effect on the scar tissue. Please note the special information regarding medical care with Silon-TEX®.

Compression therapy generally is a long-term treatment and requires the corresponding products which – if the medical indication is given – the patient him/herself applies upon the doctor's instruction.

## Properties

CICATREX® compression garment combines efficient medical care and high wearing comfort. The material's high elasticity provides for ease of putting the material on or taking it off, and it also ensures great freedom of movement.

CICATREX® is hypoallergenic, prevents inflammations in the tissue, relieves itching and provides comprehensive protection for the fresh scar.

## Indications

Damage of the skin and the skin's healing function with hypertrophic scarring or keloid formation.

- Following injuries / burns / surgical interventions
- Skin grafts
- For improvement of scar / wound healing until complete healing of the scar
- For the stabilisation of surgical results
- Medical care of lymph oedemas in special cases

## Absolute contraindications

Peripheral arterial occlusive disease (PAOD) stage III/IV; decompensated heart failure; septic phlebitis; phlegmasia coerulea dolens; acute erysipelas; acute bacterial, viral or allergic inflammations; untreated or advanced dermatoses /contact dermatitis; untreated open wounds; swelling of extremities of unknown origin; untreated malignant tumours.

Do not use Silon-TEX® in case of fissured skin or open wounds; also, do not use it for transplants which did not heal in a stable manner.

## Relative contraindications

Pronounced exudative dermatoses; malignant lymph oedema; peripheral arterial occlusive disease (PAOD) stage I/II; peripheral neuropathy associated with diabetes mellitus; sensitivity disorders of extremities; compensated heart failure; skin infections; dermatoses (e.g. blistering dermatoses); intolerance of pressure or of product ingredients; primary chronic polyarthritis. In case of open wounds in the treatment area, the wound area must first be covered with an adequate wound dressing or a bandage before the compression garment is applied.

Thuasne Deutschland GmbH accepts no liability for failure to observe these contraindications.

## Side effects

No side effects are known if properly used. In case of any negative changes during the application, please consult your doctor or the medical retailer immediately. In case of complaints in connection with the product, such as damage of the knitted fabric or lack of fit, please contact directly your medical retailer.

Please note the following if Silon-TEX® is sewn into your compression garment: Should negative changes occur, such as itching or a skin allergy due to silicone intolerance, please contact immediately your doctor or your medical retailer. Should incompatibilities regarding one or several ingredients of this product be known, please consult your doctor prior to use. If skin irritations or allergies persist, stop using Silon-TEX®. The manufacturer shall not be liable for damages / injuries which resulted due to improper handling or misuse.

## Precautions

Follow the instructions given by your doctor, by an orthopaedic technician or medical expert personnel. If you experience severe pain or discomfort, please see your doctor, an orthopaedic technician or medical expert personnel. Store at room temperature, preferably in the original packaging.

To guarantee its effectiveness and for reasons of hygiene, the product may not be passed on to other patients. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

### Directions for putting the garment on

Your medical retailer will be pleased to show you the correct application. We recommend that compression garments are permanently worn 24 hours per day.

Rough, unfiled fingernails or sharp-edged rings etc. may damage the extremely fine knitted fabric. Therefore wear rubber gloves please. We recommend our fitting aids if there are difficulties putting on the garment. In case of high UV exposure due to sun light, our skin needs to be protected. This is especially true for scar tissue. We therefore recommend to use, under your compression garment, a suitable sunscreen product in the form of creams or the like.

If Silon-TEX® is sewn into your CICATREX® compression garment, please note the following:

- Scar and the surrounding skin must be thoroughly cleaned.
- The scar tissue is to be kept dry and grease-free.
- Do not use ointments, creams, gels or lotions in combination with Silon-TEX®.
- Put on your compression garment such that the Silon-TEX® is placed directly on the scar area and fits smoothly.
- Do not use in case of skin irritations.
- Do not use in case of a known intolerance of the ingredient (high-purity silicone).
- Should skin incompatibilities occur during application, please stop using the product and consult your doctor
- Do not use Silon-TEX® on open wounds!

### Care

Compression garments should be washed daily, if possible. The garment fabric not only collects remainders of ointments and grease but also skin scales, sweat and dust. Regular washing will protect you from infections and your garment from wear. You need a spare set of garments so that you need not interrupt your compression therapy while the other set is washed and dried.

Always turn your compression garment inside out to wash it. For care of the garments, we recommend Thuasne Compression Clean Detergent Concentrate. Alternatively use a mild detergent for washing by hand or for machine wash (gentle cycle at 40° C) without optical brightener and fabric softener. In a laundry bag, compression garments are specially protected. In order to reduce the drying period, put the compression garments into a terry cloth towel, roll it up firmly and press the garment out (do not wring it out). Do not leave the moist compression garment in the wet towel. Hang up your garment loosely and leave it to air-dry. Never put it in the sun or onto a radiator to dry.

If Silon-TEX® is sewn into your compression garment, please dry the piece of garment especially well for reasons of hygiene and store it loosely as far as possible and with ventilation.

Do not expose compression garments to any damaging effects, such as: strong sunlight, intense heat, ointments, oils and greases, stain remover, gasoline and aggressive detergents.

### Material composition

Please find the exact information on the sewn-in fabric label.

Note: The sewn-in fabric label is an important component of the compression garment. If you remove this labelling from a Thuasne product, any claim for warranty, repair or replacement shall be waived.

Material composition of Silon-TEX®: Silicone layer: 100 % medical grade silicone, carrier material: 81 % polyamide, 19 % elasthan

### Storage information and duration of use

Store in a dry place and protect from sunlight. For series products, a maximum shelf life of 36 months applies. This is followed by the duration of use of 6 months for the compression garment. The hourglass pictogram shows the duration of use printed on the carton label. Customised manufacture concerns products for immediate use.

For serial sizes and custom-tailored garments, we guarantee their medical efficacy for a maximum period of wear of six months. Prerequisite for this will be the proper treatment of the compression materials (for example, their care or putting them on or taking them off). If necessary, your doctor should then prescribe new compression garments. In case of any significant change of relevant body sizes (e.g. due to a successful therapy, weight change), new measurements and a new supply may also be justified earlier already.

### Disposal

You may dispose of your medical compression garment via the residual or household waste. No special criteria exist for their elimination.

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THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
**THUASNE DEUTSCHLAND GmbH**