**THUASNE**

# Cicatrex<sup>®</sup>

Articles de compression continue

Continuous compression garments

Dauerkompressionsbandagen

Permanente drukkleding

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Conserver cette notice.

# CONTINUOUS COMPRESSION GARMENTS en

## Instructions for use

### Indications :

- Treatment of scars :
  - heat, chemical or electrical burns,
  - prevention of functional or unaesthetic complications of wound healing (keloids, hypertrophy, adhesions),
  - skin grafts.
- Maintenance of volume reduction in lymphoedema.

### Preliminary warning :

- The garment supplied may appear to be small in comparison with the body's measurements. This is because it is a compression garment, the dimensions of which are necessarily smaller than those of the body, since the objective is to compress the scar region. It is therefore quite natural for the patient to experience initial discomfort after putting it on.
- In the case of garments fastened with Velcro straps, always make sure that there is no direct contact between the Velcro and the skin (since this could cause irritation).
- The seams must always be showing, on the outside of the garment, to prevent them hurting the skin.

### When Cicatrex® should be worn ?

- Use during the immediate or late post-scar formation phase.
- Follow the instructions given by the doctor and encourage the patient to wear **Cicatrex®** as regularly as possible, day and night. **Cicatrex®** is an essential treatment to improve the appearance of scars. The patient should only stop wearing **Cicatrex®** on his/her doctor's advice.

### How to pull on and position Cicatrex® ?

- When the patient pulls on his/her garment or when you put it on for him/her, stretch the elastic fabric as much as possible so that fragile skin is not irritated while the garment is being pulled on.
- Once the garment has been put on, it may be necessary to help the fabric evenly fit all the contours of the parts of the body covered and be properly positioned without getting in the way of clothing and without being too tight.
- Pull up sleeves if they are too long ; make sure there is freedom of movement at the elbow and shoulder area, etc.

### Precautions for use :

- When using for the first few times, regularly check for the absence of :
  - oedema in the distal regions,
  - ischaemia,
  - skin ulcers or blisters under the garment (in particular, monitor zones subject to excessive pressure or friction),
  - pain.
- Thereafter, in the event of significant discomfort, pain or any unusual symptoms consult your doctor or a health professional.
- In children, continuous compression can lead to growth problems (mainly in the head area). Monitoring must be more frequent and the garments must be regularly remade to match the growth of the patient.
- The fabrics used keep the risks of allergy to a minimum ; however, exceptional cases of allergy are still possible.
- For hygiene and performance reasons, do not re-use the product for another patient.

### How to wash Cicatrex® ?

- Always wash **Cicatrex®** by hand, as often as you want, using soap or a product designed for delicate fabrics. Do not add bleach or stain-remover. Rinse thoroughly and squeeze out excess water without twisting.
- Dry flat away from any direct heat (do not dry on a radiator, for example). Do not iron.
- If possible, have two **Cicatrex®** garments of the same form available so that the patient can continue to wear one when the other is being washed.

### How to repair any tears ?

- Mend **Cicatrex®** by hand, creating a solid seam but making sure that it is as loose as possible (so that **Cicatrex®** remains elastic).
- It is also possible to adjust or repair **Cicatrex®** with a sewing machine, using "zig-zag" stitching only.

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**Thuasne Benelux B.V.**
**Thuasne Scandinavia AB**
**Thuasne Cervitex Ltd**
**Thuasne UK Ltd**
**Thuasne (Shanghai) Medical Devices Co., Ltd.**