

**THUASNE**

# MOBIDERM

<b>fr</b>	Plaque et bandage mobilisateurs pour les œdèmes .....	4
<b>en</b>	Mobilizing pad and bandage for edema .....	5
<b>de</b>	Mobilisierende Pads und Bandagen zur Behandlung von Ödemen .....	6
<b>nl</b>	Mobilisatiepad en -verband voor oedemen .....	8
<b>it</b>	Placca e benda di mobilizzazione per edemi .....	10
<b>es</b>	Placa y venda movilizadoras para edemas .....	11
<b>pt</b>	Placa e ligadura de mobilização para os edemas .....	13
<b>da</b>	Mobiliserende plade og bind til ødemer .....	14
<b>fi</b>	Mobilisoiva levy ja sidos turvotukseen .....	16
<b>sv</b>	Mobiliserande ark och bandage för ödem .....	17
<b>el</b>	Πλάκα και επίδεσμος κινητοποίησης για τα οιδήματα .....	18
<b>cs</b>	Mobilizační podložka a obinadlo pro léčbu edémů .....	20
<b>pl</b>	Arkusz i bandaż mobilizujący na obrzęki .....	21
<b>lv</b>	Mobilizējošā plātnīte un saite tūskām .....	23
<b>lt</b>	Poodinę cirkuliaciją skatinanti plokštelė ir juosta ištinusioms vietoms .....	24
<b>et</b>	Nahaaluseid kudesid mobiliseeriv padi ja side ödeemide puhuks .....	25
<b>sl</b>	Mobilizacijska blazinica in povoj za edeme .....	27
<b>sk</b>	Podložka a obváz na mobilizáciu edémov .....	28
<b>hu</b>	Ödémacsökkentő mobilizáló lap és fásli .....	30
<b>bg</b>	Мобилизираци подложки и бинтове за отоци .....	31
<b>ro</b>	Placă și bandaj de mobilizare pentru edeme .....	33
<b>ru</b>	Мобилизующие компрессы и повязки для лечения отеков .....	34
<b>hr</b>	Podložak i zavoj za imobilizaciju kod edema .....	36
<b>zh</b>	用于水肿的活动泡沫片和活动绷带 .....	37
<b>ja</b>	リンパ浮腫用活性化プレートおよび包帯 .....	38
<b>ko</b>	부종 모빌라이징 패드 및 붕대 .....	39
<b>ar</b>	رفادة وضمادة مُحركة للوذمات .....	43

<b>fr</b>	Dispositif médical	Un seul patient - à usage multiple
<b>en</b>	Medical device	Single patient - multiple use
<b>de</b>	Medizinprodukt	Einzelner Patient - mehrfach anwendbar
<b>nl</b>	Medisch hulpmiddel	Eén patiënt - meervoudig gebruik
<b>it</b>	Dispositivo medico	Singolo paziente - uso multiplo
<b>es</b>	Producto sanitario	Un solo paciente - uso múltiple
<b>pt</b>	Dispositivo médico	Paciente único - várias utilizações
<b>da</b>	Medicinsk udstyr	Enkelt patient - flergangsbrug
<b>fi</b>	Lääkinnällinen laite	Potilaskohtainen - voidaan käyttää useita kertoja
<b>sv</b>	Medicinteknisk produkt	En patient - flera användningar
<b>el</b>	ιατροτεχνολογικό προϊόν	Πολλαπλή χρήση - σε έναν μόνο ασθενή
<b>cs</b>	Zdravotnický prostředek	Jeden pacient - vícenásobné použití
<b>pl</b>	Wyrób medyczny	Wielokrotne użycie u jednego pacjenta
<b>lv</b>	Medicīniska ierīce	Viens pacients - vairākkārtēja lietošana
<b>lt</b>	Medicinos priemonė	Vienas pacientas - daugkartinis naudojimas
<b>et</b>	Meditsiiniseade	Ühel patsiendil korduvalt kasutatav
<b>sl</b>	Medicinski pripomoček	En bolnik - večkratna uporaba
<b>sk</b>	Zdravotnícka pomôcka	Jeden pacient - viacnásobné použitie
<b>hu</b>	Orvostechnikai eszköz	Egyetlen beteg esetében többször újrahasználató
<b>bg</b>	Медицинско изделие	Един пациент - многократна употреба
<b>ro</b>	Dispozitiv medical	Un singur pacient - utilizare multiplă
<b>ru</b>	Медицинское изделие	Многократное использование для одного пациента
<b>hr</b>	Medicinski proizvod	Jedan pacijent - višestruka uporaba
<b>zh</b>	醫療器械	一位患者 - 多次使用
<b>ja</b>	医療機器	一患者用 - 複数回使用
<b>ko</b>	의료 기기	한 환자에 한 해 - 여러번 사용
<b>ar</b>	جهاز طبي	لمريض واحد - متكرر الاستعمال

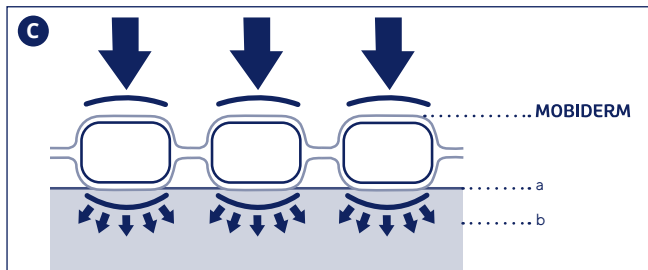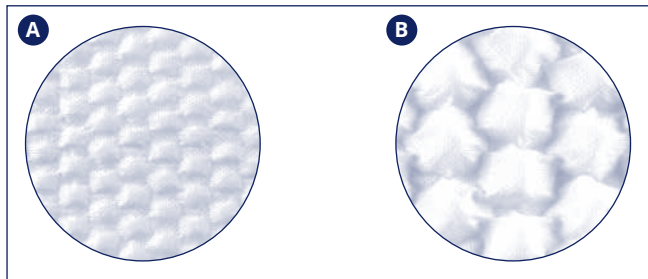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## en MOBILIZING PAD AND BANDAGE FOR EDEMA

### Description

MOBIDERM devices are a range of mobilizing devices for the treatment of edema. This range is available in several versions:

MOBIDERM bandages:

- small blocks (5 x 5 mm) **A**
- big blocks (15 x 15 mm) **B**

MOBIDERM pads:

- small blocks (5 x 5 mm) **A**
- big blocks (15 x 15 mm) **B**

The devices are composed of MOBIDERM technology consisting of foam blocks (polyurethane for the big blocks or polyethylene for the small blocks) encased between two sheets of non-woven material (polyester, acrylic adhesive mass).

MOBIDERM devices are available in different sizes of pads and bandages and in two foam block sizes to be used according to need and the body area treated.

### Mechanism of action

MOBIDERM devices are intended to be used under a compression bandage. In this way they provide subcutaneous drainage via MOBIDERM technology consisting of foam blocks positioned randomly between two sheets of non-woven material. In contact with the skin, the blocks provide a pressure differential between the contact area beneath them and their periphery, creating multi-directional circulation channels for lymph under the skin. The lymph travels from the areas pressed by the blocks towards the free areas. Efficient lymph drainage and circulation is thus promoted. **C**

Use: MOBIDERM mobilizing devices are intended for treatment of edema of the upper or lower limbs.

### Indications

MOBIDERM mobilizing devices are indicated in the treatment of acute or chronic edema (for example: lymphedema).

Target population: adult patients and children suffering from indicated pathologies.

### Fitting

-  It is recommended that your device be washed before it is used for the first time.
- Always make sure that the product is intact before each use.
- A cotton bandage or protective bandage must be placed underneath the MOBIDERM device.
- It is possible to place cushioning devices (such as Varico, foam bandage) underneath MOBIDERM devices.
- It is possible to cut the devices to fit the area to be treated according to need. It is

recommended that the devices be cut with scissors between 2 rows of foam blocks.

- Check that there are no folds in the device once in place.

#### Application of MOBIDERM bandages:

- Apply the bandage circularly along the length of the limb without pressure with an overlap of one row of blocks between each turn. At the joint, we recommend an overlap of 2 rows of foam blocks in order to maintain the position of the device during various body movements.
- Cut off surplus bandage if necessary.
- Then apply the compression bandage according to the manufacturer's recommendations.

#### Application of MOBIDERM pads:

- Locate the area to be treated then cut the pad according to the surface shape chosen.
- Apply the pad to the area to be treated without pressure.
- Then apply the compression bandage according to the manufacturer's recommendations.

#### **Contraindications**

- Skin infections of the limb or acute inflammation.
- Known allergies to any of the components.
- Septic thrombosis.
- Severe peripheral neuropathy of the limb.

#### Lower limbs:

- Lower limb peripheral arterial disease (PAD) with ankle brachial index (ABI) < 0.6.
- Decompensated heart failure.
- Advanced diabetic microangiopathy.
- Phlegmasia cerulea dolens (painful blue oedema with arterial compression).
- Extra-anatomic bypass.

#### Upper limbs:

- Brachial plexus pathology.
- Vasculitis of the extremities.

#### **Precautions**

- Follow the advice of a health care professional regarding treatment duration and methods.
- Do not use the product directly on damaged skin.
- Fully cover any skin lesions with an appropriate dressing before fitting the device.
- Application of this device can cause the appearance of marks on the skin. These marks are benign and temporary; do not stop treatment without medical advice.
- If you experience significant hindrance, discomfort, pain, skin damage, infection, unusual sensations (example: tingling), coloring of the extremities, or change in performance, remove the garment and consult a health care professional.
- If the edema worsens, consult a health care professional.

- Due to the constantly changing nature of this condition, we recommend regular check-ups with a health care professional to ensure the device is still suitable for you.
- Do not use when swimming. Do not expose to a heat source or seawater.
- Do not apply creams before applying the device as they may damage the product.
- If you suffer from advanced venous disease or ulcers, consult a health care professional for advice on a suitable full-leg compression garment.
- For reasons of hygiene, safety and performance, do not share your garment with anyone else. The product should not be reused for another patient.

#### **Undesirable side-effect**

- Skin reactions: blisters, itching, marks, burns and redness.
- Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

#### **Care/Maintenance**

To ensure your MOBIDERM devices last as long as possible and to prevent infection, it is important to wash them every day, or after each use.

- Soak the device in tepid soapy water, rub with a clean sponge and squeeze out excess water.
- Do not use chlorine-based detergents or harsh products.
- Air dry, away from direct heat (e.g. radiator, sun, etc.).
- Do not tumble-dry.
- Do not iron.

#### **Disposal**

Check local regulations for how to dispose of MOBIDERM devices.

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©Thuasne - 2019801 (2020-03)