THUASNE®

Argicalm

fr	Pack Chaud Froid - Coussin d'argile réutilisable.....	6
en	Hot and cold pack - Reusable clay pack	7
de	Wärme- und Kältepackung - Wiederverwendbare Lehmpackung.....	9
nl	Hot/cold pack - Herbruikbare kleipakking	11
it	Cuscinetto Caldo Freddo - Cuscinetto di argilla riutilizzabile	13
es	Paquete Caliente Frio - Paquete de arcilla reutilizable	14
pt	Pack Frio Quente - Embalagem de argila reutilizável.....	16
da	Kold/Varm Pakning - Genbrugelig lerpakning.....	18
fi	Lämpö-kylmäpakkaus - Kestokäyttöinen lämpösavipakkaus.....	20
sv	Varm kall förpackning - Återanvändningsbar lerdyna.....	21
el	Πακ Θερμό Ψυχρό - Επαναχρησιμοποιούμενη κομπρέσα αργίλου.....	23
cs	Teplý a studený zábal - Opakovaně použitelný jílový polštářek.....	25
pl	Okład gorący i zimny Okład termiczny wielokrotnego użytku zawierający glinę... 27	27
lv	Karstā/aukstā komprese Vairākkārt lietojama mālu komprese	28
lt	Šildomasis-šaldomasis paketas - Daugkartinis molio paketas.....	30
et	Pakend kuum/külm - Korduvkasutatav savimähis	32
sl	Toplo hladni komplet - Glinena obloga za večkratno uporabo.....	33
sk	Obklad plnený prírodným ílom na úľavu od bolesti Opakovane použiteľný obklad plnený prírodným ílom.....	35
hu	Hideg-meleg csomag - Újrafelhasználható agyagpakolás	37
bg	Топъл и студен пакет - Глинен възглавница за многократна употреба.....	39
ro	Pachet pentru terapie caldă/rece - Pachet de argilă reutilizabil	41
ru	Термальный пакет - Многоцветная глиняная подушка	42
hr	Hladno-topli paket - Jastučić s glinom za višekratnu uporabu	44
zh	热套装-可循环使用粘土热袋.....	46
ar	وسادة مُسَخَّنة ومُبرَّدة وسادة طينية للاستخدام المتكرر.....	49

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucţiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - إرشادات الاستعمال

1

19 x 11 cm

**2**

26 x 13 cm

**3**

30 x 20 cm



**4**

43 x 17 cm



fr	Thermothérapie	Cryothérapie
en	Thermotherapy	Cryotherapy
de	Wärmetherapie	Kältetherapie
nl	Thermotherapie	Cryotherapie
it	Termoterapia	Crioterapia
es	Termoterapia	Crioterapia
pt	Termoterapia	Crioterapia
da	Termoterapi	Kryoterapi
fi	Termoterapia	Kryoterapia
sv	Termoterapi	Kryoterapi
el	Θερμοθεραπεία	Κρυοθεραπεία
cs	Termoterapie	Kryoterapie
pl	Termoterapia	Krioterapia
lv	Termoterapija	Krioterapija
lt	Šilumos terapija	Šalčio terapija
et	Termoteraapia	Krüoteraapia
sl	Topla terapija	Hladna terapija
sk	Termoterapia	Kryoterapia
hu	Termoterápia	Krioterápia
bg	Термотерапия	Криотерапия
ro	Termoterapie	Crioterapie
ru	Термотерапия	Криотерапия
hr	Termoterapija	Krioterapija
zh	温热理疗	冷疗
ar	معالجة بالحرارة	معالجة بالبرد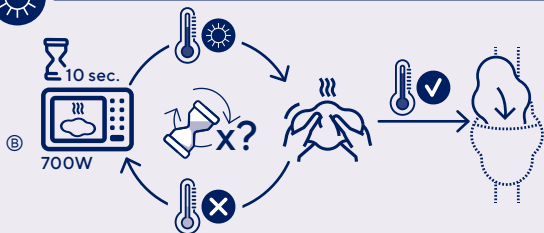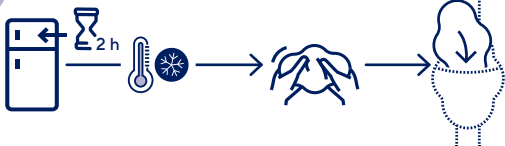

Taille*	1	2	3	4
	x 7	x 8	x 8	x 11

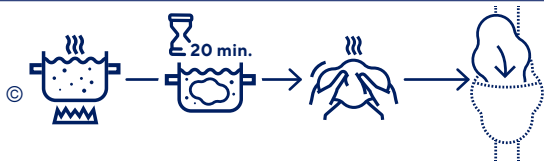* Size - Größe - Maat - Taglia - Talla - Tamanho - Størrelse - Koko - Storlek - Μέγεθος - Velikost - Rozmiar - Izmērs - Dydis - Suurus - Velikost - Velkost' - Méret - Размер - Mărire - Размер - Veličina - 尺码 - المقياس

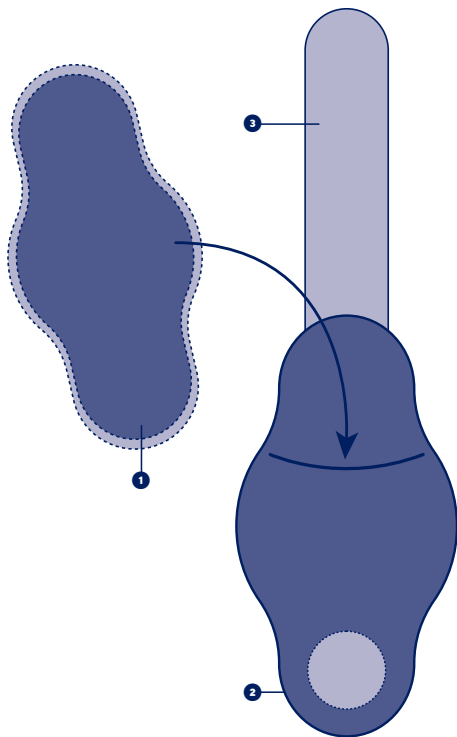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Conserver cette notice.

en

HOT AND COLD PACK - REUSABLE CLAY PACK

Description/Destination

Reusable clay pack. ❶

Removable protected sleeve and comfortable.

Velvet effect. ❷

Self-adhesive elastic fastening strap. ❸

The device is intended only for the treatment of the indications listed in adults, adolescents and children.

Composition

Cotton - elastane - polyester - clay - PVC

Properties/Mode of action

Use cold:

Soothes pain, reduces inflammation, bruises and edema/swelling.

Use hot:

Soothes pain, increases blood flow, relaxes.

Indications

Use cold:

Sprains, strains, tendinitis, migraines/headache, eye pain, toothache/dental pain, bruises, arthritis, maxillofacial surgery and eye surgery.

Use hot:

Back pain, Contractures/muscle and articular stiffness, cramps, torticollis (stiff neck), abdominal pain.

Contraindications

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

In case of diabetes or vascular and circulatory problems, consult a doctor before using this product.

Do not use the cold mode in case of hypersensitivity to cold.

Do not use with analgesic and/or anesthetic products (such as creams, balms, lotions, liniments...).

Precautions

In the event of discomfort, significant hindrance, pain, abnormal sensations, remove the device and consult a healthcare professional.

Monitor the use of the product in children or in people with cognitive disabilities.

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Do not apply directly to the skin, use the provided sleeve.

External use only, do not ingest contents.

In case of accidental contact, rinse thoroughly, drink water if swallowed.

Do not heat up the Argicalm clay pack using a microwave or a bain-marie if it is not already at room temperature.

Do not heat Argicalm clay packs beyond the duration and associated power indicated in the present instructions for use: the clay pack could be damaged or even destroyed.

If the power of the microwave is unknown, use the lowest power.

Be careful not to burn yourself when handling the Argicalm clay pack with bare hands in case of hot water or microwave use.

If the Argicalm clay pack is too hot, wait until it cools down before using it in its sleeve.

Always check clay pack temperature before application, prolonged application beyond provided indications may cause injuries including burns.

Do not place excessive weight, sit, or lie down on the pack.

Do not use after the expiry date indicated on the clay pack.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Use cold:

Instructions Ⓐ :

Remove the clay pack Argicalm from its bag. Place it flat in a freezer compartment (-20°C) for at least 2 hours before use.

For more comfort, massage/knead the Argicalm clay pack to make it more flexible before inserting it in its sleeve.

If the Argicalm clay pack is too cold or too solid, leave it for a few minutes at room temperature before using it.

To avoid the risk of cold injuries, do not apply on body for more than 20 min and wait at least 20 min before applying again.

Use hot:

Instructions for use in the microwave Ⓑ :

- Remove the clay pack Argicalm from its bag.
- Place it flat at room temperature in the microwave and follow the heating protocol.
- Massage the clay pack to spread warmth evenly before inserting it in its sleeve.

Instructions for use in a bain-marie © :

- Heat water to boiling point.
- Make sure the volume is sufficient to fully immerse the clay pack.
- Turn off the heat source.
- Remove the clay pack Argicalm from its bag.
- Insert the Argicalm clay pack at room temperature into the hot water for from 5 to a maximum of 20 minutes (depending on the clay pack weight) until it reaches the required temperature.

Care/Maintenance

Hand washable with soft detergent. Do not dry clean. Do not apply solvent or alcohol to the pack. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Due to the natural origin of the clay contained in Argicalm, the appearance of the clay pack may change during the life of the product (colored dots or marble-like areas).

This change in color has no impact on the function or performance of the Argicalm clay packs.

Disposal

Dispose of in accordance with local regulations.

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THUASNE UK Ltd
Thuasne SAS – SIREN/RCS Nanterre 542 091 186