THUASNE®

Sous-couche compressive pied - Chaussettes

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Conserver cette notice.

en

FOOT COMPRESSION UNDERLAYER - SOCKS. FOOT COMPRESSION SOCKS.

Description/Use

The product is a sock-shaped undergarment designed to be worn under a compression device (Biflex Self Adjust - Calf).

It consists of two distinct zones:

- Compression zone (20-30 mmHg): extends from the base of the toes to above the malleolus.
- Non-compressive zone: skin-protection underlayer extending from the malleolus to below the knee.

Composition

Polyamide - elastane - cotton.

Properties/Mode of action

Skin protection as the first protective layer under compression devices.

Compression around the foot.

Indications

Management of edema (venous, lymphatic and caused by injury).

Chronic venous insufficiency (CEAP classification stage C3 to C6).

Contraindications

Do not use in case of major dermatological conditions on an affected limb.

Do not use in the event of known allergy to any of the components.

Do not use in case of lower limb peripheral arterial disease (PAD) with ankle-brachial index (ABI) < 0.6.

Do not use in the event of decompensated heart failure.

Do not use in the event of advanced diabetic microangiopathy (for a compression > 30 mmHg).

Do not use in case of severe peripheral neuropathy of the affected limb.

Do not use in the event of phlegmatia coerulea dolens (painful blue phlebitis with arterial compression) of the affected limb.

Do not use in case of extra-anatomical bypass of the affected limb.

Do not use in the event of septic thrombosis.

Precautions

Strictly follow your healthcare professional's prescription and recommendations to use.

These devices are designed to be worn day and night.

The risk/benefit ratio and the adequate level of pressure must be regularly re-assessed by a healthcare professional in the event of:

- PAD with ABPI between 0.6 and 0.9
- Advanced peripheral neuropathy
- Weeping or eczematous dermatitis.

For hygiene, safety and performance reasons, do not re-use the product for another patient. In the event of discomfort, significant restriction, pain, variation in limb volume, deterioration of skin condition, infection, unusual sensations, change in the colour of the extremities, or changes in the performance of the product, remove the product and seek the advice of a healthcare professional.

Do not use products on the skin (creams, ointments, oils, gels, patches, etc.) before applying the device as this could damage the product.

Do not apply the product directly to injured skin or open wounds without a suitable dressing. When using this device with other medical devices not manufactured by Thuasne®, consult a healthcare professional.

Take care not to damage the device when fitting it, particularly with your nails.

Do not use when swimming.

Side-effects

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity or cutaneous dryness.

Any serious device-related incidents occurring should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Slip your foot into the sock until the heel is correctly positioned.

Pull up to below the knee and do not fold the sock over itself.

Ensure that the device is free of creases to avoid any risk of discomfort or injury.

Care

It is recommended to wash the product every day, or after each use.

The devices are designed for daily use over a period of 6 months (i.e., 90 washes). Machine washable at 40 °C. Use a laundry net if possible. Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Dry away from any direct heat source (radiator, sun, etc.). Do not tumble-dry. Dry flat.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2001201 (2025-09)
THUASNE UK Ltd
Thuasne SAS – SIREN/RCS Nanterre 542 091 186