**THUASNE®**

# Epi-GO

<b>de</b>	Ellenbogenstütze gegen Epicondylitis .....	5
<b>en</b>	Anti-epicondylitis elbow brace .....	6
<b>fr</b>	Coudière anti-épicondylite .....	7
<b>nl</b>	Anti-epicondylitis elleboogbrace .....	8
<b>it</b>	Gomitiera antiepicondilite .....	9
<b>es</b>	Codera antiepicondilitis .....	10
<b>pt</b>	Cotoveleira antiepicondilite .....	11
<b>da</b>	Albueunderlag mod tennisalbue .....	13
<b>fi</b>	Kyynärtuki epikondyliitin hoitoon .....	14
<b>sv</b>	Armbågsskydd mot epikondylit .....	15
<b>el</b>	Καλύπτρα αγκώνα κατά της επικονδυλίτιδας .....	16
<b>cs</b>	Loketní ortéza proti epikondylitidě .....	17
<b>pl</b>	Orteza stawu łokciowego przeciwko entezopatii nadkłytkcia .....	18
<b>lv</b>	Antiepicondilīta elkoņa pārsējs .....	19
<b>lt</b>	Alkūnės įtvaras epikondilitui gydyti .....	20
<b>et</b>	Epikondüliidi küünarnukitugi .....	22
<b>sl</b>	Komolčna blazinica proti epikondilitisu .....	23
<b>sk</b>	Antiepicondylická bandáž .....	24
<b>hu</b>	Teniszkönyök elleni könyökrögzítő .....	25
<b>bg</b>	Налакътник срещу епикондилит .....	26
<b>ro</b>	Cotieră anti-epicondilită .....	27
<b>ru</b>	Налокотник против воспаления надмыщелка .....	29
<b>hr</b>	Anti-epikondilitis štitnik za laktove .....	30
<b>zh</b>	抗上髌炎护肘 .....	31
<b>ar</b>	واقى مرفق مضاد لالتهاب اللقيمة .....	33

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - إرشادات الاستعمال


	0	21 - 23 cm
	1	23 - 25 cm
	2	25 - 27 cm
	3	27 - 29 cm
	4	29 - 31 cm
	5	31 - 34 cm





<b>de</b>	Umfang 5 cm unter dem Ellbogen
<b>en</b>	Circumference 5 cm under the elbow
<b>fr</b>	Circonférence 5 cm sous le coude
<b>nl</b>	Omtrek 5 cm onder de elleboog
<b>it</b>	Circonferenza 5 cm sotto il gomito
<b>es</b>	Circunferencia de 5 cm bajo el codo
<b>pt</b>	Circunferência 5 cm abaixo do cotovelo
<b>da</b>	Omkreds 5 cm under albuen
<b>fi</b>	Ympärysmitta 5 cm kyynärpään alapuolelta
<b>sv</b>	Omkrets 5 cm under armbågen
<b>el</b>	Περίμετρος 5 εκ. κάτω από τον αγκώνα
<b>cs</b>	Obvod 5 cm pod loktem
<b>pl</b>	Obwód 5 cm poniżej łokcia
<b>lv</b>	Apkārtmērs 5 cm zem elkoņa
<b>lt</b>	Apimtis 5 cm žemiau alkūnės
<b>et</b>	Küünarnuki all übermõõt 5 cm
<b>sl</b>	Obseg 5 cm pod komolcem
<b>sk</b>	Obvod 5 cm pod lakťom
<b>hu</b>	Kerület könyök alatt 5 cm-rel
<b>bg</b>	Обиколка 5 cm под лакътя
<b>ro</b>	Circumferință de 5 cm sub cot
<b>ru</b>	Окружность 5 см ниже локтя
<b>hr</b>	Opseg 5 cm ispod lakta
<b>zh</b>	肘部下方周长5 厘米
<b>ar</b>	محيط بمقدار 5 سم أسفل المرفق

<b>de</b>	Halt
<b>en</b>	Support
<b>fr</b>	Maintien
<b>nl</b>	Fixatie
<b>it</b>	Sostegno
<b>es</b>	Sujeción
<b>pt</b>	Manutenção
<b>da</b>	Støtte
<b>fi</b>	Tuki
<b>sv</b>	Stöd
<b>el</b>	Συγκράτηση
<b>cs</b>	Stabilizace
<b>pl</b>	Podtrzymywanie
<b>lv</b>	Atbalsts
<b>lt</b>	Palaikymas
<b>et</b>	Hooldus
<b>sl</b>	Oprijem
<b>sk</b>	Podpora
<b>hu</b>	Támasz
<b>bg</b>	Поддръжка
<b>ro</b>	Suținere
<b>ru</b>	Стабилизация
<b>hr</b>	Potpورا
<b>zh</b>	稳定
<b>ar</b>	الدعم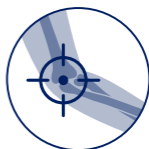

<b>de</b>	Kompression	Propriozeptive Wirkung	Lokaler Druck
<b>en</b>	Compression	Proprioceptive effect	Targeted pressure
<b>fr</b>	Compression	Effet proprioceptif	Pression localisée
<b>nl</b>	Compressie	Proprioceptieve werking	Lokale druk
<b>it</b>	Compressione	Effetto propriocettivo	Pressione localizzata
<b>es</b>	Compresión	Efecto propioceptivo	Presión localizada
<b>pt</b>	Compressão	Efeito proprioceptivo	Pressão localizada
<b>da</b>	Kompression	Proprioceptisk effekt	Lokalt tryk
<b>fi</b>	Kompressio	Liikeaistiin kohdistuva vaikutus	Paikallinen paine
<b>sv</b>	Kompression	Proprioceptiv effekt	Lokaliserat tryck
<b>el</b>	Συμπίεση	Αποτέλεσμα ιδιοδεκτικότητας	Εντοπισμένη πίεση
<b>cs</b>	Komprese	Proprioceptivní účinek	Lokalizovaný tlak
<b>pl</b>	Kompresja	Propriocepcja	Ucisk miejscowy
<b>lv</b>	Kompresija	Proprioceptīvs efekts	Lokalizēts spiediens
<b>lt</b>	Kompresija	Propriocepcinis poveikis	Konkrečios vietos suspaudimas
<b>et</b>	Kompressioon	Proprioceptiivne toime	Paikne surve
<b>sl</b>	Kompresija	Proprioceptivni učinek	Lokaliziran pritisk
<b>sk</b>	Kompresia	Proprioceptívny účinok	Presne zacielený tlak
<b>hu</b>	Szorítás	Proprioceptív hatás	Lokalizált nyomás
<b>bg</b>	Компресия	Проприоцептивен ефект	Локализирано налягане
<b>ro</b>	Compressie	Efect proprioceptiv	Presiune localizată
<b>ru</b>	Компрессия	Проприоцептивный эффект	Локализованное давление
<b>hr</b>	Kompresija	Učinak proprioceptivnog vježbanja	Lokalizirani pritisak
<b>zh</b>	压紧	本体感	局部施压
<b>ar</b>	الضغط	مفعول استقبال الحس العميق	ضغط موضعي

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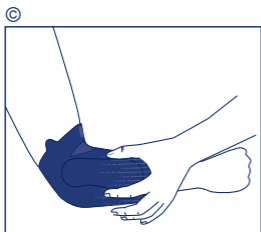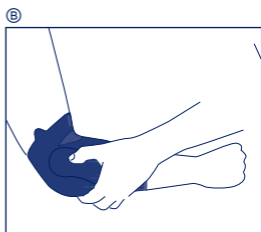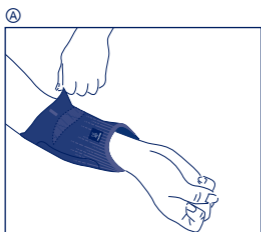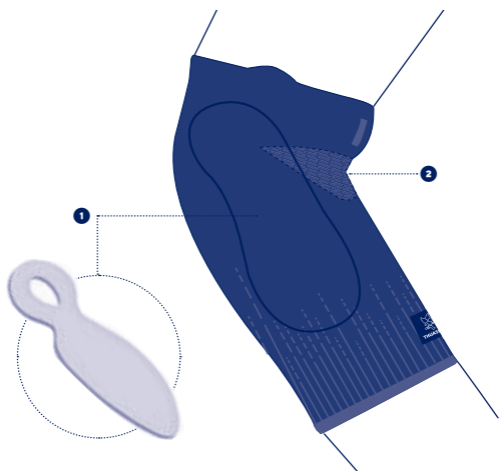## **ANTI-EPICONDYLITIS ELBOW BRACE**

### **Description/Destination**

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

This device is an elastic proprioceptive elbow brace providing targeted pressure to relieve tendon pain.

This device can be used as a proprioceptive support for rehabilitation and particularly when resuming sports activities.

### **Composition**

Textile components: polyamide (81%) - elastane (13%) - polyester (6%)

Non-textile components: polyurethane - polyethylene foam

### **Properties/Mode of action**

Elbow support with anatomically-shaped inserts. ❶

Anatomically-shaped knit for a good fit. ❷

### **Indications**

Treatment of lateral or medial epicondylitis (tennis/golfer's elbow).

Prevention of elbow tendinopathy recurrence.

### **Contraindications**

Do not use the product if the diagnosis has not been confirmed.

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

### **Precautions**

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

### **Undesirable side-effect**

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

## Instructions for use/Application

Slip on the elbow support, positioning the soft blue knitted zone in the crease of the elbow, with the pull tabs upwards. Ⓐ

Position the inserts on either side of the elbow and centre the holes over the epicondyles (medial and lateral). Ⓑ Ⓒ

## Garment care

Product can be washed in accordance with the conditions shown on this instruction leaflet and on the label.

Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Machine washable at 30 °C (delicate programme). Do not tumble-dry. Do not iron. Do not dry clean. Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.).

## Storage

Store at room temperature, preferably in the original packaging.

## Disposal

Dispose of in accordance with local regulations.

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Placera inläggen på båda sidor om armbågen och centrera vecken i höjd med epikondylerna (medial och lateral). Ⓑ ©
**THUASNE UK Ltd**
Thuasne SAS – SIREN/RCS Nanterre 542 091 186