
THUASNE

Genu PFS PROMASTER

- de **Gebrauchsanweisung - Knieorthese mit Gelenkschienen und Patella-Zugsystem**
- en **Instructions for use - Knee orthosis with hinges and patella traction system**
- fr **Consignes d'utilisation - Genouillère articulée avec système de traction de la rotule**
- it **Istruzioni per l'uso - Ortesi articolata per il ginocchio con sistema di trazione rotulea**
- es **Instrucciones de utilización - Ortesis de rodilla con flejes articulados y sistema de tracción patelar**
- pt **Instruções de uso - Ortótese para o joelho com hastas articuladas e sistema de tração da rótula**
- nl **Gebruiksaanwijzing - Knieorthese met scharnierspalken en patella-treksysteem**
- da **Brugsanvisning - Knæ-ortese med ledskinner og patella-træksystem**
- sv **Bruksanvisning - Knäortos med ledskenor och patelladragsystem**
- fi **Käyttöohje - Polvitukiside nivelkiskoilla ja polvilumpion vetojärjestelmällä**
- el **Οδηγίες χρήσης - Κηδεμόνας γόνατος με ενσωματωμένες αρθρώσεις και σύστημα έλξης της επιγονατίδας**
- cs **Návod k použití - Kolenní ortéza s kloubovými výztuhami a patelárním tažným systémem**
- pl **Sposób użytkowania - Orteza stawu kolanowego z szynami przegubowymi i systemem podciągania rzepki**
- et **Kasutusjuhend - Põlveortoos liigendsiinide ja põlvekedra tõmbesüsteemiga**
- lv **Lietošanas instrukcija - Ceļgala ortoze ar locītavas šinām un ceļa kaula savilces sistēmu**
- lt **Naudojimo instrukcija - Kelio įtvartas su lankstais ir kelio girnelės įtempimo sistema**
- sk **Návod na použitie - Kolenná ortéza s klbovými dlahami a trakčným systémom po luxácii pately (vyklbení jablčka)**
- sl **Navodila za uporabo - Kolenska orteza z vodili za zgibe in vlečnim sistemom za pogačico**
- hu **Használati utasítás - Térd ortézis oldalsínekkal és patellafeszítő rendszerrel**
- ro **Instrucțiuni de utilizare - Orteză pentru genunchi cu atelă pentru articulație și sistem de întindere a patelui**
- bg **Инструкция за употреба - колянна ортеза с шарнирни шини и пристягащ механизъм на пателата**
- hr **Upute za uporabu - Ortoza za koljeno s udlagama za zglob i sustavom za povlačenje patele**

Größe ¹	Oberschenkelumfang ²		Wadenumfang ³	
	cm	inch	cm	inch
XXS	35 – 38	13 ¾ - 15	25 – 28	9 ¾ - 11
XS	38 – 41	15 - 16	28 – 31	11 - 12 ¼
S	41 – 44	16 - 17 ¼	31 – 34	12 ¼ - 13 ½
M	44 – 47	17 ¼ - 18 ½	34 – 37	13 ¼ - 14 ¾
L	47 – 50	18 ½ - 19 ¾	37 – 40	14 ¾ - 15 ¾
XL	50 – 53	19 ¾ - 21	40 – 43	15 ¾ - 17
XXL	53 – 56	21 - 22	43 – 46	17 - 18

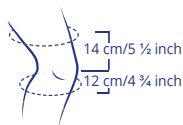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Knee orthosis with hinges and patella traction system

Purpose:

The Genu PFS PROMASTER is a knee orthosis with a patella pad and hinges for correction and support of the patella glide path. The orthosis may only be used for knee treatment.

Properties:

- Anatomically shaped patella pad
- Hinges integrated at the sides
- Traction system for correction and support of the patella glide path
- 3D knit fabric covering the patella
- Highly flexible 3D knit fabric in the hollow of the knee
- Elasticated knit bandage
- Latex-free

Indications:

- Patello-femoral pain syndrome
- Severe degenerative changes in the patella glide path
- Post-operative therapy after MPFL and lateral release
- Patellar lateralisation
- Patellar instability
- Patellar luxation and subluxation
- Retropatellar arthrosis

Contraindication:

Do not apply the product in direct contact with broken skin.

Side effects:

No side effects are as yet known for correct use.

Wearing instruction:

The traction system should be loosened or the orthosis should be removed during longer periods of rest, such as when sitting for a long time or sleeping.

Directions for putting on the orthosis:

Before you put it on:

- Please open the calf strap and loosen the patella traction system first.
- To avoid damage to the knit fabric, please make sure that the Velcro® straps are always fastened to the provided Velcro® pads when putting it on and taking it off.

Putting on the orthosis:

- Grasp the orthosis at the top. To make it easier to put it on, grasp the interior donning loops with your index fingers.
- Turn the orthosis by approximately 90°. Then step into the orthosis with your foot extended and pull it up over your leg.
- Please make sure that the patella pad is placed correctly around the patella (= kneecap).

- Then close the calf strap. To avoid constriction, please make sure that the strap is not pulled too tightly. Loosen it if required.
- To adjust the patella traction system individually, simply undo the Velcro® fastening on the inside of the leg, pull it upwards along your body with slight tension - until the required pressure is reached - and then re-fasten it to the Velcro® pad.
- Opening the Boa® fastener: Stretch your leg, press on the top part of the fastener (on the rivet) using your thumb and grasp into the recessed grip between the two pulling cables from below using your index finger. Then pull the lower part of the fastener upwards until it opens and comes loose.
- Closing the Boa® fastener: Grasp into the recessed grip from below using your index finger and hold the fastener between your thumb and index finger. Then hook the lower part into the upper part and press it downwards.

Before you take it off:

- Please open the calf strap and loosen the patella traction system. Then grasp into the orthosis from below and pull it off downwards.

Precautions:

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Washing instructions:

- Do not use detergents, fabric softeners or aggressive products (products containing chlorine)
- Squeeze out excess water
- Dry away from heat sources

Material composition:

78 % polyamide, 14 % elastane, 5 % cotton, 2 % polyurethane, 1 % PES polyester, EVA foam

Explanation of the reinforced proprioceptive effect:

Orthoses in the PROMASTER range have been developed to improve the therapeutic effect - and, in particular, proprioception - on the joint. To achieve this, they activate special skin receptors on the joint and improve the latter's muscle support, thanks to innovative padding and an optimised compression effect.

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Boa® is a registered trademark of Boa Technology Inc.
THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
Thuasne UK Ltd
THUASNE DEUTSCHLAND GmbH