THUASNE

Genu OA | Genu OA Plus

- de **Gebrauchsanweisung - Knieorthese zur Entlastung und Führung bei Gonarthrose**
en **Instructions for use - Off-loading knee brace for the treatment of knee osteoarthritis**
- fr **Consignes d'utilisation - Genouillère de décharge pour le traitement de la gonarthrose**
- it **Istruzioni per l'uso - Ginocchiera di scarico per il trattamento della gonoartrosi**
es **Instrucciones de utilización - Órtesis con sistema de descarga de rodilla para el tratamiento de osteoartritis**
- pt **Instruções de uso - Joelheira de descarga para o tratamento da artrose do joelho**
nl **Gebruiksaanwijzing - Ontlastende orthese voor de behandeling van gonartrose**
da **Brugsanvisning - Støttende knæbeskytter til behandling af slidgigt i knæ**
sv **Bruksanvisning - Avlastande knäortos för behandling av artros**
fi **Käyttöohje - Kuormitusta keventävä polvituki polviartroosin hoitoon**
el **Οδηγίες χρήσης - Επιοργανίδα αποφόρτισης για τη θεραπεία της οστεοαρθρίτιδας**
- cs **Návod k použití - Ortéza kolenní s nastavitelným rozsahem pro léčbu gonartrózy**
pl **Sposób użytkowania - Orteza odciążająca staw kolanowy do stosowania podczas leczenia stanów zapalnych i zmian zwyrodnieniowych**
- et **Kasutusjuhend - Koormust leevendav põlveortoos gonartroosi taastusraviks**
lv **Lietošanas instrukcija - Ceļa ortoze gonartrozes ārstēšanai**
lt **Naudojimo instrukcija - Apkrovą mažinantis kelio įtvaras gonartrozei gydyti**
sk **Návod na použitie - Odlahčovací kolenná ortéza na liečbu gonartrózy (artrózy kolenného kĺbu)**
- sl **Navodila za uporabo - Kolenska razbremenilna ortoza za zdravljenje artroze kolena**
- hu **Használati utasítás - Tehermentesítő ortézis térdartrózis kezeléséhez**
ro **Instrucțiuni de utilizare - Orteză de descărcare a genunchiului, pentru tratarea gonartrozei**
- bg **Инструкция за употреба - Стабилизираща колянна ортеза за лечение на гонартроза**
- hr **Upute za uporabu - Ortoza za koljeno za otpuštanje i vođenje kod osteoartritisa**

1

2

3

4


5

6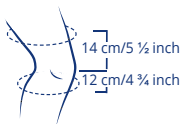

**Genu OA
(35047)**

Größe ¹	Oberschenkelumfang ²		Wadenumfang ³	
	cm	inch	cm	inch
XS	38–41	15 – 16	28–31	11 – 12 ¼
S	41–44	15 – 17 ¼	31–34	12 ¼ – 13 ½
M	44–47	17 ¼ – 18 ½	34–37	13 ½ – 14 ¾
L	47–50	18 ½ – 19 ¾	37–40	14 ¾ – 15 ¾
XL	50–53	19 ¾ – 21	40–43	15 ¾ – 17
2XL	53–57	21 – 22 ½	43–46	17 – 18
3XL	57–61	22 ½ – 24	46–49	18 – 19 ½
4XL	61–66	24 – 26	49–52	19 ½ – 20 ½

links oder rechts⁴**Genu OA Plus
(35048)**

Größe ¹	Oberschenkelumfang ²		Wadenumfang ³	
	cm	inch	cm	inch
XS	38–45	15 – 17 ¾	28–31	11 – 12 ¼
S	41–48	15 – 18 ⅞	31–34	12 ¼ – 13 ½
M	44–51	17 ¼ – 20 ⅞	34–37	13 ½ – 14 ¾
L	47–54	18 ½ – 21 ¼	37–40	14 ¾ – 15 ¾
XL	50–57	19 ¾ – 22 ½	40–43	15 ¾ – 17
2XL	53–61	21 – 24	43–46	17 – 18
3XL	57–65	22 ½ – 25 ⅞	46–49	18 – 19 ½
4XL	61–70	24 – 27 ½	49–52	19 ½ – 20 ½

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Off-loading knee brace for the treatment of knee osteoarthritis

Purpose:

The Genu OA (Plus) is a knee orthosis with pulling system and hinge to achieve an off-loading effect and guidance of one compartment based on the 3-point leverage system. The orthosis may only be used for knee treatment

Properties:

- Adjustable 3-point leverage system with inelastic straps to provide alignment support and off-loading of the affected compartment
- Integrated hinge with flexion and extension stops
- Anatomic knit delivering homogeneous compression
- Fine and support knit at the popliteal fossa
- Silicone threads at the top of the brace, optimising holding of the orthosis on the leg
- Silicon Patella ring
- Latexfree

Indications:

- Mild to moderate uni-compartmental Osteoarthritis (Grade I to III)
- Uni-compartmental knee conditions requiring load reduction
- Postoperative and/or conservative off-loading effect of a single compartment

Contraindication:

- Do not apply the product in direct contact with broken skin or superficial wounds
- For patient with severe arterial insufficiency or severe varicosities that could result in skin at risk with regular compressive brace wear
- Allergy to component

Side effects:

No side effects are as yet known for correct use.

Directions for putting on the orthosis

Please note that the orthosis can be used as follows:

- Medial arthrosis on the right/lateral arthrosis on the left
- Medial arthrosis on the left/lateral arthrosis on the right

Before you put it on:

- First loosen all the straps
- To avoid damage to the knit fabric, please make sure that the Velcro pads are always fastened to the designated Velcro surfaces when putting it on and taking it off.

Fitting, putting on and taking off the orthosis:

For optimum use of the device, we recommend wearing it on the bare skin (except in the case contraindicated).

Important:

For an optimum fit of the joint it should be adjusted to the patient.

To avoid that the Velcro pieces come off, please make sure that they are all attached properly and firmly to the Velcro straps.

When fitting the orthosis, always make sure that the two intersecting Velcro straps (crossover Velcro straps) (Fig.1) are always positioned centrally during adjustment!

Check the spacing and position of the two crossover Velcro straps. The length of the two ends of each crossover Velcro strap protruding from the crossover strap cover must be identical.

- Grasp the orthosis at the top.
- Step into the orthosis with your foot extended and then pull it up over your leg. To make it easier to put on, turn the orthosis by approx. 90° beforehand.
- Make sure that your knee cap is positioned in the centre of the patella ring.
- (Only for Genu OA Plus) Now close the magnetic fastener on the upper edge of the orthosis. The length of the band can be adjusted, if necessary. This is possible by cutting the band to size at the rear end. In order to still vary the tension if necessary, the end of the strap should reach over the entire length of the hook and loop part.

Start with the two rear straps. First fit the lower leg and then the upper leg strap.

- To adjust the length of the two rear straps, it is necessary to adjust the length of the end opposite the Velcro strap intersection which has a straight cut edge (Fig. 2).
- To adapt the strap, open the Velcro pad and shorten the strap to the desired length (Fig.3). Then close the Velcro pad and fix it on the strap.
- The Velcro pad should be positioned centrally on the length of the strap so that the tension can be increased if necessary.
- If creases form during strap adjustment, the angle of each strap can be adjusted and positioned individually by opening the two rear hook and loop openings on the crossover Velcro strap (Fig. 4) (round end).

Then adjust the two front straps. First fit the lower leg and then the upper leg strap.

- To adjust the length of the two front straps, it is necessary to adapt the length of the end of the strap which is directly located on the crossover Velcro strap (Fig.1).
- To adapt the strap, open the Velcro pad (Fig. 5) and shorten the strap to the desired length.
- The Velcro pad should preferably be positioned centrally on the length or near the fastening loop with the magnetic fastener so that the tension can be increased if necessary.
- To close the strap, grasp into the loop on top of the magnetic fastener (Fig. 6) and pull the end of the strap towards the Velcro pad with the magnetic fastener so that the magnetic fastener can find itself and close automatically.
- If creases form during strap adjustment, the angle of each strap can be adjusted and positioned individually by opening the two front hook and loop openings on the crossover Velcro strap (Fig. 5) (round end).

Extension and flexion adjustment:

- The orthosis comes equipped with a 0° extension stop
- To adjust the desired extension or flexion, it is necessary to take the joint out of the joint pocket at the upper end on the inside.
- Flexion settings: 20°, 40°
- Extension settings: 0°, 10°

Re-fitting:

- To change the tension while wearing the orthosis, it is only necessary to change the position of the Velcro ends of the straps on the straps.
- The tension of the strap on the upper edge of the orthosis may be changed by moving the band on the hook and loop area. In this case, the strap must extend at least up to the weld seam on half of the hook and loop part.
- If the lower end of the orthosis presses into the calf, the articulated arm can be bent outwards at the distal end appropriately.

Before you take it off:

- To loosen the strap system, please open the magnetic fasteners of the two front straps and then pull on the two rear straps.
- Then grasp into the orthosis from below and pull it off downwards.

Precautions:

Follow the advice of the healthcare practitioner who prescribed or supplied the product. Wearing this product, as with any compression product, could, in some persons, cause cutaneous reactions (redness, erythema). Should this occur, it is preferable to readjust or remove the product and ask the professional who delivered the product for advice. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Washing instructions:

- Take off the rigid hinge before washing.
- Fasten Velcro tabs before washing.
- Machine wash at 30 °C, delicate cycle, preferably in a wash bag.
- Do not use detergents, fabric softeners or aggressive products (products containing chlorine)
- Squeeze out excess water.
- Allow to dry in the ambient air, remote from any source of heat.

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THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
Thuasne UK Ltd
THUASNE DEUTSCHLAND GmbH