THUASNE®

Manu-GO

de	Orthese zur Stabilisierung des Handgelenks	5
en	Wrist stabilising splint	6
fr	Attelle de stabilisation du poignet	7
nl	Stabilisatiebrace voor de pols	8
it	Tutore stabilizzatore per polso	10
es	Férula estabilizadora del pulgar	11
pt	Tala de estabilização do punho	12
da	Stabiliserende skinne til håndleddet	13
fi	Stabiloiva rannetuki	15
sv	Stabiliseringskena för handleden	16
el	Νάρθηκας σταθεροποίησης του καρπού	17
cs	Ortéza pro stabilizaci zápěstí	19
pl	Stabilizująca orteza nadgarstka	20
lv	Plauksta locītavas stabilizācijas saite	21
lt	Riešo stabilizavimo įtvaras	22
et	Randme stabiliseerimislahas	24
sl	Opornica za stabilizacijo zapestja	25
sk	Stabilizačná dlaha na zápästie	26
hu	Csuklórögzítő	27
bg	Шина за стабилизация на китка	28
ro	Atelă stabilizatoare pentru încheietura mâinii	30
ru	Стабилизирующая шина для запястья	31
hr	Udlaga za stabilizaciju ručnog zgloba	32
zh	腕部稳定夹板	34
ar	جبيرة لتحقيق استقرار الرسغ	37

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - إرشادات الاستعمال

	1	14 - 15 cm
	2	15 - 16 cm
	3	16 - 17 cm
	4	17 - 18 cm
	5	18 - 19 cm
	6	19 - 20 cm

de	Handgelenkumfang
en	Wrist circumference
fr	Tour de poignet
nl	Polsomtrek
it	Circonferenza del polso
es	Contorno de la muñeca
pt	Perímetro do punho
da	Håndleddets omkreds
fi	Ranteen ympärysmitta
sv	Handledens omkrets
el	Περίμετρος καρπού
cs	Obvod zápěstí
pl	Obwód nadgarstka
lv	Plaukstas locītavas apkārtmērs
lt	Riešo apimtis
et	Randme ümbermõõt
sl	Obseg zapestja
sk	Obvod zápästia
hu	Csukló kerülete
bg	Обиколка на китката
ro	Circumferința încheieturii
ru	Обхват запястья
hr	Obujam ručnog zgloba
zh	腕围
ar	محيط الرسغ

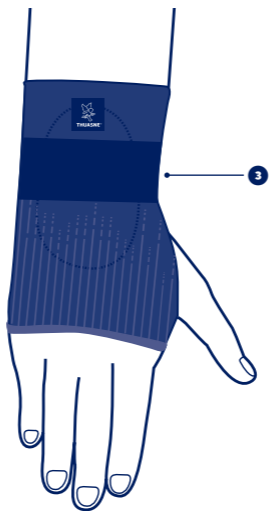
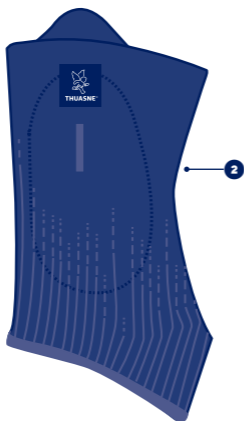
de	Stabilisierung
en	Stabilization
fr	Stabilisation
nl	Stabilisatie
it	Stabilizzazione
es	Estabilización
pt	Estabilização
da	Stabilisering
fi	Stabiloi
sv	Stabilisering
el	Σταθεροποίηση
cs	Stabilizace
pl	Stabilizacja
lv	Stabilizēšana
lt	Stabilizavimas
et	Stabiliseerimine
sl	Stabiliziranje
sk	Stabilizácia
hu	Stabilizálás
bg	Стабилизация
ro	Stabilizare
ru	Стабилизация
hr	Stabilizacija
zh	稳定
ar	الاستقرار

de	Kompression	Propriozeptive Wirkung
en	Compression	Proprioceptive effect
fr	Compression	Effet proprioceptif
nl	Compressie	Proprioceptieve werking
it	Compressione	Effetto propriocettivo
es	Compresión	Efecto propioceptivo
pt	Estabilização	Efeito propiocetivo
da	Kompression	Proprioceptisk effekt
fi	Kompressio	Liikeaistiin kohdistuva vaikutus
sv	Kompression	Proprioceptiv effekt
el	Συμπίεση	Αποτέλεσμα ιδιοδεκτικότητας
cs	Komprese	Proprioceptivní účinek
pl	Kompresja	Propriocepcja
lv	Kompresija	Proprioceptīvs efekts
lt	Kompresija	Propriocepčinis poveikis
et	Kompressioon	Proprioceptiivne toime
sl	Kompresija	Proprioceptivni učinek
sk	Kompresia	Proprioceptívny účinok
hu	Szorítás	Proprioceptív hatás
bg	Компресия	Проприоцептивен ефект
ro	Compresie	Efect proprioceptiv
ru	Компрессия	Проприоцептивный эффект
hr	Kompresija	Učinak proprioceptivnog vježbanja
zh	压紧	本体感
ar	الضغط	مفعول استقبال الحس العميق

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WRIST STABILISING SPLINT

Description/Destination

This device is a knitted wrist splint providing joint stabilisation.
The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Textile components: polyamide (75%) - polyester (13%) - elastane (8%) - latex (4%)
Non-textile components: polyurethane - aluminium - polyethylene foam

Properties/Mode of action

- Stabilisation of the wrist via a rigid, malleable and anatomical palmar stay. ❶
- Anatomically-shaped knit ensuring proprioception.
- Comfort zone at the ulnar head (foam pad). ❷
- Removable strap to adjust compression level. ❸

Indications

- Wrist sprain.
- Wrist tendinopathies.
- Post-operative, post-traumatic residual pain and/or edema.
- Wrist osteoarthritis.

Contraindications

- Do not use the product if the diagnosis has not been confirmed.
- Do not apply the product in direct contact with broken skin.
- Do not use in the event of known allergy to any of the components.

Precautions

- Verify the product's integrity before every use.
- Do not use the device if it is damaged.
- Choose the appropriate size to fit the patient, referring to the size chart.
- It is recommended that a healthcare professional supervises the first application.
- Strictly comply with your healthcare professional's prescription and recommendations for use.
- In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

If necessary, shape the palmar stay.

Slip on the splint, making sure that the thumb is correctly positioned in the opening. Position the ulnar head in the hole of the foam insert.

If additional pressure is required, add the removable strap.

Attach the strap onto the self-fastening surface on the inside of the splint, then wrap the strap around the wrist.

Removal: to remove the splint, pull simultaneously the knit and the stay.

Garment care

Product can be washed in accordance with the conditions shown on this instruction leaflet and on the label. Remove stay before washing. Replace in the original location before next use. Close the self-fastening tabs before washing. Machine washable at 30°C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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Composition

Composants textiles : polyamide (75%) - polyester (13%) - élasthanne (8%) - latex (4%)

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©Thuasne - 2052601 (2024-03)
THUASNE UK Ltd**