

THUASNE

Ortel P


fr	Ceinture pelvienne	5
en	Pelvic belt	5
de	Becken-Stützgürtel	6
nl	Bekkenbrace	7
it	Cintura pelvica	8
es	Faja pélvica	9
pt	Cinta pélvica	9
da	Bækkenbælte	10
fi	Lantiovyö	11
sv	Bäckenbälte	12
el	Ζώνη πυελικής χώρας	13
cs	Pánevní pás	14
pl	Orteza miednicowa	14
lv	Iegurņa josta	15
lt	Dubens diržas	16
et	Vaagnavöö	17
sl	Medenični pas	18
sk	Panvový pás	18
hu	Medence öv	19
bg	Колан за таз	20
ro	Centură pelviană	21
ru	Тазовый бандаж	22
hr	Pojas za zdjelicu	23
zh	骨盆支撐帶	23
ar	حزام للحوض	25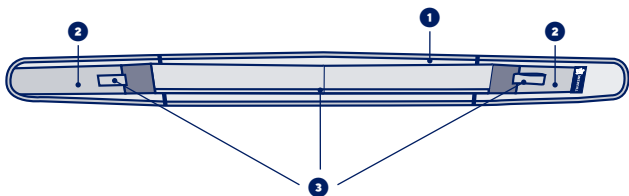

(A) (B)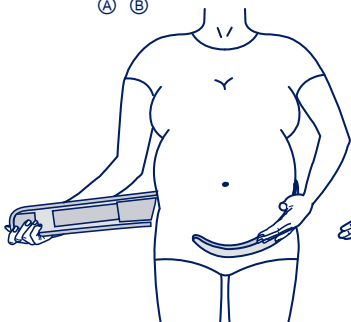

(C)

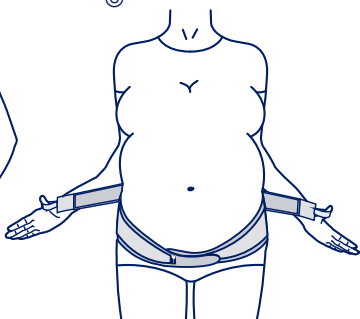(D)

fr

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Description/Destination

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PELVIC BELT

Description/Intended users

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Reinforced elastic fabric. ①

Comfort terry-towel lining.

Abdominal panels with self-fastening surface. ②

Additional strap with thumb-loops. ③

Textile components: coated elastane - polyamide - polyester.

Non-textile components: polyoxymethylene.

Properties/Mode of action

The device ensures strong support and adaptable compression to the pelvis thanks to the elastic fabric and the additional strap.

Indications

Pregnancy-related pelvic girdle pain (posterior pelvic pain, sacro-iliac pain, symphyseal pain).

Contraindications

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

Precautions

Verify the product's integrity before every use.

Wear the device above thin clothing.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

Strictly comply with your healthcare professional's prescription and recommendations for use.

Do not wear the product in a medical imaging machine.

Do not wear the product while sleeping.

In the event of discomfort, significant hindrance, pain, abnormal sensations, remove the device and consult a healthcare professional.

It is recommended to adequately tighten the device to achieve support without excessive compression.

Undesirable side-effects

This device can cause skin reactions (redness, itching, irritations...).

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Position the belt around the pelvis. **A**

The belt must be placed as low down on the hips as possible,

The inside label must be positioned upwards.

Fasten the belt, right side over left side, adjusting to the desired compression. **B**

For an adapted support, you can use the additional strap.

Pull the thumb-loops on the additional strap simultaneously and evenly. **C**

Attach the self-fastening parts onto the centre of the abdominal panels of the belt. **D**

The level of support can be adjusted as required throughout the day.

Care

Close the self-fastening tabs before washing.

Machine washable at 40°C (delicate programme). Do not machine wash the device more than 10 times. Do not dry clean. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not tumble-dry. Do not iron. Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). Dry flat.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

This medical device is a regulated health product with CE marking

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©Thuasne - 2023302 (2022-06)
THUASNE UK Ltd
Thuasne SAS – SIREN/RCS Nanterre 542 091 186 – capital 1 950 000 euros