THUASNE®

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Conservser cette notice.

en

PROTECTIVE LINER - SOCKS. ACCESSORY FOR COMPRESSION MEDICAL DEVICES

Description/Destination

The protective sock is an accessory that can be used with the following medical devices manufactured by Thuasne®: Biflex Self Adjust - Calf, Mobiderm® Autofit, Mobiderm® made-to-measure patient - lower limb models, Mobiderm® pads and bandages. The device is designed to protect the skin when wearing compression products or garments.

Composition

Polyamide - elastane.

Properties/Mode of action

The device protects the skin under a compression garment.

Contraindications

Do not use in case of major dermatological conditions on an affected limb. Do not use in the event of known allergy to any of the components.

Precautions

Strictly comply with your healthcare professional's prescription and recommendations for use.

For hygiene, security and performance reasons, do not re-use the product for another patient.

In the event of discomfort, significant restriction, pain, variation in limb volume, deterioration of skin condition, infection, unusual sensations, change in the colour of the extremities, or changes in the performance of the product, remove the product and seek the advice of a healthcare professional.

Do not use products on the skin (creams, ointments, oils, gels, patches...) before applying the device, it could damage the product.

Do not apply the product directly to an open wound without a dressing.

When using this accessory with other medical devices not manufactured by Thuasne®, seek the advice of a healthcare professional.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity or cutaneous dryness.

Any serious device-related incidents occurring should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Put on the protective underlay up to the knee. Put the compression garment over it. The protective underlay can be folded back over the garment.

Garment care

Machine washable at 40°C. Avoid machine washing the device more than 90 times. Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Do not tumble-dry. Dry flat.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2056901 (2025-01)
THUASNE UK Ltd
Thuasne SAS - SIREN/RCS Nanterre 542 091 186