THUASNE

Rhizo-pro

- de Gebrauchsanweisung - Daumensattelgelenk-Orthese
en Instructions for use - Wrist/thumb orthosis
fr Consignes d'utilisation - Orthèse statique poignet pouce
it Istruzioni per l'uso - Ortesi per il pollice e il metacarpo
es Instrucciones de utilización - Órtesis de la articulación trapeciometocarpiana
pt Instruções de uso - Órtese para a articulação trapeziometocarpiana
nl Gebruiksaanwijzing - Duim-handwortel-Orthese
da Brugsanvisning - Tommelfingerortose
sv Bruksanvisning - Tumortos
fi Käyttöohje - Peukalon tyvinivelen tukiside
el Οδηγίες χρήσης - Νάρθηκας επιπλοειδούς άρθρωσης αντίχειρα
cs Návod k použití - Ortéza na palec ruky
pl Sposób użytkowania - Orteza stawu siodełkowatego kciuka
et Kasutusjuhend - Pöidlapõhimiku liigese ortoos
lv Lietošanas instrukcija - Plaukstpamata un īkšķa locītavas ortoze
lt Naudojimo instrukcija - Riešo ir nykščio įtvaras
sk Návod na použitie - Ortéza na palec ruky
sl Navodila za uporabo - Orteza za sedlasti sklep palca
hu Használati utasítás - Hüvelykujj nyeregízületi ortézis
ro Instrucțiuni de utilizare - Orteză metacarpo-falangiană pentru policeu
bg Инструкция за употреба - Ортеза за седловидната става на палеца
hr Upute za uporabu - Ortoza za sedlasti zglob palca

Größe ¹	 Handgelenkumfang ²		 Daumenbreite ³	
	cm	inch	cm	inch
1	12,0 - 18,5	4 ¾ - 7 ¼	1,7 - 2,2	⅔ - ⅘
2	16,5 - 22,0	6 ½ - 8 ⅔	2,2 - 2,6	⅘ - 1 ⅕

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45 % Polyamid, 32 % Elasthan, 23 % Polyester

Wrist/thumb orthosis**Purpose:**

Rhizo-pro is a thumb orthosis used for immobilisation. The orthosis may only be used for thumb treatment.

Mode of action:

Reliable immobilisation of the carpal and first carpo-metacarpal (CMC I) joints as well as the metacarpo-phalangeal (MCP I) joint of the thumb. The anatomically correct shape of the product ensures a comfortable, secure fit.

Indications:

Arthrosis/arthritis, distortions, injuries (trauma), instabilities of the ligaments, postoperative

Contraindications:

Skin trauma or disease in the thumb or wrist area

Side Effects:

No side effects are as yet known for correct use.

Precautions:

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Fitting instructions:

1. Open the touch-and-close fastener on the support.
 2. Place the thumb fully into the orthosis.
 3. Wrap the wrist strap around your wrist joint and secure.
 4. Wrap the thumb strap around your thumb and secure.
- The orthosis should lend firm support without cutting in.

Care Instructions:

To clean the orthosis, remove the plastic shell from the textile part. Close Velcro straps before washing. Clean the plastic shell with a damp cloth and a little washing-up liquid.

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Thuasne UK Ltd
THUASNE DEUTSCHLAND GmbH