THUASNE


Silistab® Achillo

fr	Chevillère élastique proprioceptive pour tendon d'Achille.....	5
en	Proprioceptive elastic ankle brace for Achilles tendon.....	6
de	Elastische propriozeptive Knöchelbandage für Achillessehne	7
nl	Proprioceptieve elastische enkelbrace voor achillespees.....	9
it	Cavigliera elastica propriocectiva per tendine di Achille.....	10
es	Tobillera elástica propioceptiva para tendón de Aquiles.....	12
pt	Tornoeleira elástica proprioceptiva para tendão de Aquiles.....	13
da	Proprioceptisk, elastisk ankelbindtil achillessene.....	15
fi	Proprioseptiivinen joustava nilkkatuki akillesjanteelle.....	16
sv	Proprioceptiv elastisk ankelortos för hälsenan.....	17
el	Ελαστική ιδιοδεκτική επιστραγαλίδα για αχίλλειο τένοντα.....	18
cs	Elastická proprioceptivní kotníková ortéza na Achillovu šlachu.....	20
pl	Elastyczna proprioceptywna orteza skokowo-stopowa z wkładkami odciążającymi.....	21
lv	Proprioceptīvs elastīgs potītes pārsējs Ahileja cīpslai.....	23
lt	Tamprus proprioceptiją gerinantis čiurnos įtvaras Achilo sausgyslei.....	24
et	Proprioseptiivne elastne hüppeliigesetugi Achilleuse kõõluse jaoks.....	25
sl	Elastična proprioceptivna opornica za Ahilovo tetivo.....	27
sk	Elastická proprioceptívna členková ortéza na Achillovu šľachu.....	28
hu	Proprioceptív rugalmas bokarögzítő az Achilles-ínhez.....	30
bg	Проприоцептивна еластична шина за глезен за ахилесовото сухожилие.....	31
ro	Glezniară elastică proprioceptivă de protecție a tendonului lui Ahile.....	32
ru	Эластичный проприоцептивный ортез для разгрузки ахиллова сухожилия.....	34
hr	Elastični proprioceptivni steznik za gležanj za Ahilovu tetivu.....	35
zh	跟腱彈力本体感护踝.....	37
ar	دعامة كاحل مرنة متعلقة باستقبال الحس العميق لوتر العرقوب.....	39

	1	19 - 21 cm
	2	21 - 23 cm
	3	23 - 25 cm
	4	25 - 27 cm
	5	27 - 30 cm

fr	Tour de cheville
en	Ankle circumference
de	Knöchelumfang
nl	Omvang van de enkel
it	Giro caviglia
es	Circunferencia del tobillo
pt	Perímetro do tornozelo
da	Anklens omkreds
fi	Nilkanympärys
sv	Runt fotleden
el	Περίμετρος αστραγάλου
cs	Obvod kotníku
pl	Obwód kostki

lv	Potītes apkārtmērs
lt	Kulkšnies apimtis
et	Pahkluu ümbermõõt
sl	Obseg gležnja
sk	Obvod členka
hu	Boka kerülete
bg	Обиколка на глезена
ro	Circumferința gleznei
ru	Окружность голени
hr	Opseg gležnja
zh	腳踝尺寸
ar	محيط الكاحل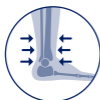

fr	Compression	Effet proprioceptif	Décharge
en	Compression	Proprioceptive effect	Off-loading
de	Kompression	Propriozeptive Wirkung	Entlastung
nl	Compressie	Proprioceptieve werking	Losmaken
it	Compressione	Effetto propioceettivo	Scarico
es	Compresión	Efecto propioceptivo	Descarga
pt	Compressão	Efeito propioceptivo	Alívio do peso
da	Kompression	Proprioceptisk effekt	Aflastning
fi	Kompressio	Liikeaistiin kohdistuva vaikutus	Vapautus
sv	Kompression	Proprioceptiv effekt	Avlastning
el	Συμπίεση	Αποτέλεσμα ιδιοδεκτικότητας	Αποφόρτιση
cs	Komprese	Proprioceptivní účinek	Snížení zátěže
pl	Kompresja	Propriocepcja	Odciążenie
lv	Kompresija	Proprioceptīvs efekts	Atbrīvošana
lt	Kompresija	Propriocepcinis poveikis	Apkrovos sumažinimas
et	Kompressioon	Proprioceptiivne toime	Koormuse vähendamine
sl	Kompresija	Proprioceptivni učinek	Razbremenitev
sk	Kompresia	Proprioceptívny účinok	Uvoľnenie
hu	Kompresszió	Propriocepció hatás	Tehermentesítés
bg	Компресия	Проприоцептивен ефект	Освобождаване
ro	Compresie	Efect proprioceptiv	Descărcare
ru	Компрессия	Проприоцептивный эффект	Разгрузка
hr	Kompresija	Učinak proprioceptivnog vježbanja	Rasterećenje
zh	压紧	本体感	减压
ar	الضغط	مفعول استقبال الحس العميق	تخفيف ثقل

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en

PROPRIOCEPTIVE ELASTIC ANKLE BRACE FOR ACHILLES TENDON

Description/Use

This device is a proprioceptive elastic ankle brace providing joint support.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Polyamide - elastane - polyester - polyurethane - thermoplastic elastomer - ethylene vinyl acetate - presence of natural latex.

Properties/Mode of action

Elastic compressive knitting (class 3 - 20-36 mmHg / 271-48 hPa).

Insert at the Achilles tendon to protect the tendon and absorb shocks.

2 stackable pairs of heelpieces:

- to offload the Achilles tendon,
- to balance the other leg,
- to allow 3 different offloading heights depending on the pathology.

Tabs to make it easier to pull on.

Indications

Conservative treatment of Achilles tendon rupture.

Post-operative pain and edema of the Achilles tendon.

Prevention and treatment of Achilles tendon injuries: tendonitis, tenosynovitis, bursitis, achillodynia and Haglund's disease.

Prevention of sports injuries of the Achilles tendon.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin or an open wound without an adequate dressing.

Do not use in the event of known allergy to any of the components.

History of venous or lymphatic disorders.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Possible risk of venous thrombosis.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Two pairs of heelpieces are supplied with the ankle brace. Make sure you use the same number of heelpieces (1 or 2) on both sides, following the advice of the physician who prescribed the device.

Slip the ankle brace over the foot, positioning the insert over the Achilles tendon.

If necessary, superimpose the two heelpieces, placing heelpiece Ⓑ (the slimmest one) on top of heelpiece Ⓐ (the highest one).

Insert the base heelpiece (Ⓐ) or the two heelpieces (Ⓐ and Ⓑ) into the shoe of the foot wearing the ankle brace. Repeat the same operation for the other shoe.

Care

Product can be washed in accordance with the conditions shown on this instruction leaflet and on the label.

Machine washable at 30°C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2023502 (2021-09)